# medi Shoulder action

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medi

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## fitting instruction

medi


These donning instructions show you one way to put on the shoulder orthosis in just a few steps. The instructions apply equally to both the left and right arms.

Please consult your doctor about what you should pay attention to when donning – for example, how far you can lift or strain your arm.


① Insert the affected arm through the sleeve of the orthosis (Fig. 1).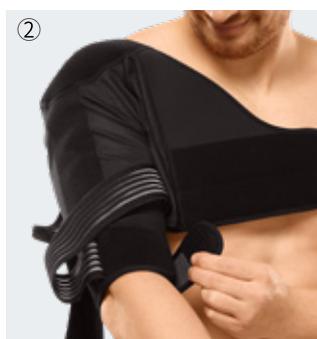


② Fasten the small strap on the inner side of the upper arm (Fig. 2).

③ Close the two ends of the chest strap on the front, just below the chest (Fig. 3).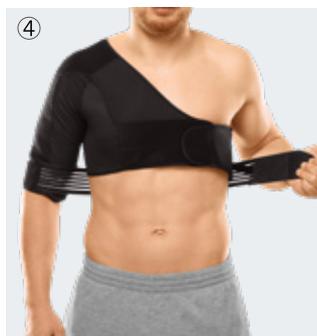

④ Now tighten the elastic strap running across the back and attach it to the chest strap as well. You can further restrict the freedom of movement by tugging lightly (Fig. 4).

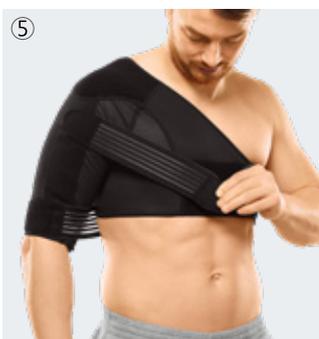⑤ Take the second additional strap and attach it to the front of the upper arm. Slide it diagonally over the chest area while tugging lightly and attach it to the chest strap (Fig. 5).

### medi tip:

Check the correct fit of the entire shoulder orthosis in front of the mirror.

### Donning videos

You can watch the donning videos via QR code or short URL [medi.biz/shoulder-action](https://medi.biz/shoulder-action)

Intended purpose: medi shoulder action is a shoulder joint orthosis with definable movement limitation

4 064984 701250 0032308/08.2020